

Solutions for better hearing

dry star UV 2

# audifon dry star UV 2





## audifon Solutions for better hearing

audifon's dry star UV2 is the perfect tool for comfortable drying and cleaning of your electronic devices. Special UV-C light kills 99.9% of common germs and bacteria. dry star UV2 is easy to use and works full automatically.

### Product Specification

- all functions are fully processor controlled
- effective drying by convective flow of warm and dry air
- two-circute heating with automatic room and temperature modulation
- drying temperature range about  $50^{\circ}\text{C} \pm 3^{\circ}\text{C}$
- touch sensor button: 4 hour comfort-drying starts
- hold sensor button for more than 3 seconds: stop program manually
- programm finishes automatically after 4 hours
- safety interruption to avoid direct UV-C contact and temperatures beyond  $60^{\circ}\text{C}$
- hygienic cleansing with UV-C light at wavelength of 253.7nm, kills 99.9% of germs and bacteria, UV-C intensity  $>400 \mu\text{W}/\text{cm}^2$
- low energy consumption: mean power 2.5 W, max. power (UV-C) 4 W, stand-by  $< 0.1 \text{ W}$

### Power

- Switching power supply (class V) with USB-socket + USB cable 1.0 m
- Input: 100–240 V, AC 50/60 Hz
- Output: 5 V, DC, 1 A, Adapter EU/UK/US/AUS; available upon request

### Size

- L 123 mm x W 90mm x H 36 mm

### Weight

- net: 166 g

### Scope of Delivery

- dry star UV2
- USB power supply, USB cable
- Hardcover
- user guide

### Warranty

2 years after delivery

### Distribution

audifon GmbH & Co. KG  
Werner-von-Siemens-Straße 2  
99625 Kölleda  
Germany

Phone +49-3635-4056-590  
Fax +49-3635-4056-589

contact@audifon.com



Manufacturer:  
flow-med GmbH · Neckarweg 7 · 71139 Ehningen · Germany  
Phone +49 7034-2774-718 · info@flow-med.com · www.flow-med.de

www.audifon.com

